<b>DE</b> Manuelle Milchpumpe Gebrauchsanweisung .....	<b>TR</b> Manuel süt pompası Kullanım kılavuzu .....
<b>EN</b> Manual breast pump Instructions for use .....	<b>RU</b> Ручной молокоотсос Инструкция по применению .....
<b>FR</b> Tire-lait manuel Mode d'emploi .....	<b>PL</b> Ręczny laktator Instrukcja obsługi .....
<b>ES</b> Extractor de leche manual Instrucciones de uso .....	
<b>IT</b> Tiralatte manuale Istruzioni per l'uso .....	

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**Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.**

## Dear customer,

Thank you for choosing a product from our range. Our name is synonymous with high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,  
Your Beurer team

## Getting to know your device:

The Beurer BY 15 breast pump set allows nursing women to express their breast milk easily and in the comfort of their own homes. The breast pump also allows you to store breast milk in the bottle, for example if you will be travelling or away from your baby for an extended period of time.

## Table of contents

<b>1. Included in delivery</b> .....	<b>11</b>	7.2 Operating the breast pump .....	16
<b>2. Signs and symbols</b> .....	<b>12</b>	7.3 Storing the breast milk .....	16
<b>3. Intended use</b> .....	<b>12</b>	7.4 Feeding breast milk to the baby .....	17
<b>4. Warnings and safety notes</b> .....	<b>13</b>	<b>8. Accessories and replacement parts</b> .....	<b>17</b>
<b>5. Device description</b> .....	<b>14</b>	<b>9. Disposal</b> .....	<b>17</b>
<b>6. Initial use</b> .....	<b>14</b>	<b>10. What if there are problems?</b> .....	<b>17</b>
6.1 Assembling the breast pump .....	15	<b>11. Technical specifications</b> .....	<b>18</b>
6.2 Cleaning and sterilising the breast pump ..	15	<b>12. Warranty/service</b> .....	<b>18</b>
<b>7. Usage</b> .....	<b>15</b>		
7.1 Tips to ensure a better milk flow .....	15		

## 1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the contents and contact your retailer or the specified Customer Service address.

- Manual breast pump
- Pump attachment with silicone membrane and silicone valve
- 1x silicone cushion (standard size)
- 1x bottle
- 1x screw plug for bottle
- Silicone teat
- Bottle cap
- Bottle holder
- 2x adapters for Avent and NUK bottles
- These instructions for use

## 2. Signs and symbols

The following symbols appear in the instructions for use and on the device.

	<b>WARNING</b> Warning notice indicating a risk of injury or damage to health
	<b>IMPORTANT</b> Safety note indicating possible damage to the device/accessories
	<b>Product information</b> Note on important information
	Read the instructions
	Manufacturer
 Operating	Permissible operating temperature and humidity
 Storage	Permissible storage temperature and humidity
	Indicates that the packaging contains products that are intended to come into contact with food, pursuant to Regulation (EC) No 1935/2004.
	This product satisfies the requirements of the applicable European and national directives
	Medical device
	Batch designation
	Dispose of packaging in an environmentally friendly manner

## 3. Intended use

### Indication/clinical benefits

The Beurer BY 15 breast pump set allows nursing women to express their breast milk easily and in the comfort of their own homes. If not enough breast milk is expressed from women who have recently given birth, this can lead to the painful build-up of milk, which in the worst-case scenario could cause inflammation of the nipples (mastitis). This build-up of milk can be prevented through targeted expressing of breast milk using the breast pump.

In addition, breast pumps help to alleviate cracked or sore nipples. You can use the integrated stimulation function to draw out flat or inverted nipples, making it easier to then express the breast milk.

### Contraindications

For hygiene reasons, the breast pump may only be used on one person. Do not share the breast pump with other nursing women. The breast pump must only be used on the female breast. Never use the breast pump if feeling drowsy or while sleeping. This device must not be used by people with reduced physical, sensory or mental skills or a lack of experience or knowledge.

### Target group:

The breast pump must be used exclusively for expressing breast milk from nursing women. Do not use the breast pump on animals! The breast pump is only intended for domestic/private use, not for com-

mercial use. This device may only be used for the purpose for which it is designed and in the manner specified in the instructions for use. Any form of improper use can be dangerous.

## 4. Warnings and safety notes

### WARNING

#### Safety notes

- Keep packaging material away from children. There is the risk of suffocation.
- Keep small parts out of reach of children.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.
- The breast pump must not come into contact with hot surfaces or sharp-edged objects.
- Do not use any additional parts that are not recommended by the manufacturer or offered as accessories.
- For hygiene reasons, the breast pump may only be used on one person. Do not share the breast pump with other nursing women.
- The breast pump must only be used on the female breast.
- Never use the breast pump if feeling drowsy or while sleeping.
- Never use the breast pump in the bath or the shower.
- Repairs must only be carried out by Customer Services or authorized suppliers. Under no circumstances should you open or repair the breast pump yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will invalidate the warranty.
- Never place the breast pump in the microwave.
- Never heat up breast milk in the microwave as it may be heated unevenly and could burn your child when they drink it.
- This device must not be used by people with reduced physical, sensory or mental skills or a lack of experience or knowledge.
- The breast pump does not require calibration.
- The breast pump and accessories must not be modified.

### IMPORTANT

#### Instructions for transporting the bottle

- When transporting breast milk in the bottle (e.g. when on the move or travelling), make sure that the screw plug is always screwed on tightly.

### WARNING

#### Child safety

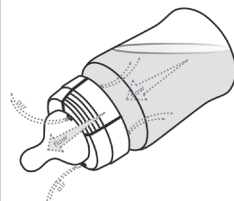
- This device may only be used under adult supervision.
- Feeding from the teat continuously and for long periods can lead to tooth decay.
- Use the teat for feeding, not as a pacifier.
- Always check the temperature of the milk before feeding.
- Inspect the device before each use and pull the teat in all directions.
- Dispose of the teat at the first sign of damage or wear and tear.
- Do not expose the teat to direct sunlight or heat and do not leave in disinfectant (“sterilising solution”) for longer than recommended as this may damage the teat.
- Keep the teat away from sweet substances as these may cause tooth decay.
- Do not use any abrasives or anti-bacterial cleaning agents.
- Sterilise all parts that come into contact with the breast milk for five minutes in boiling water.
- Clean the device before each use.
- Make sure that the temperature of the liquid in the bottle never exceeds 50°C.

## 5. Device description

1. Manual pump handle (2 levels)	7. Teat
2. Silicone membrane	8. Bottle cap
3. Silicone valve	9. Screw plug for bottle
4. Silicone cushion (standard size)	10. Adapter for NUK (the breast pump can also be used with this bottle if desired)
5. Bottle (180 ml)	11. Adapter for Avent (the breast pump can also be used with this bottle if desired)
6. Bottle holder	

Bottle with anti-colic system: regulates the pressure in the bottle and prevents air from entering your baby's stomach to reduce burping.

## 6. Initial use

### **i** Note

Check that the cardboard packaging of the breast pump has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the breast pump or accessories and that all packaging material has been removed. If you have any doubts, do not use the contents and contact your retailer or the specified Customer Service address.

## 6.1 Assembling the breast pump

Once the breast pump has been removed from the packaging, check the plastic containers in particular for cracks.

<p>1. Screw the pump attachment onto the bottle. Ensure the pump attachment is firmly secured so that no liquid can escape.</p>	
<p>2. Attach the handle for the manual pump including the silicone membrane by pressing the silicone membrane firmly onto the pump attachment until it clicks into place.</p>	

## 6.2 Cleaning and sterilising the breast pump

### **WARNING**

**Clean and sterilise the separate parts of the breast pump that come into direct contact with the breast milk individually before first use and any further use.**

To sterilise the breast pump correctly, proceed as follows:

1. Unscrew the pump attachment from the bottle.
2. Remove the silicone membrane and silicone valve from the pump attachment.
3. Sterilise the bottle and pump attachment (with silicone membrane and silicone valve) for five minutes in boiling water or using a steam steriliser. Do not use the breast pump until all individual parts are dry. Do not use any antibacterial cleaning products, as these may damage the plastic on the breast pump.

## 7. Usage

### 7.1 Tips to ensure a better milk flow

- Choose a location to use the pump where you can relax especially well.
- Choose a time to use the pump when you can relax especially well.
- Having your child nearby or looking at a picture of your child can have a positive effect on the milk flow.
- If feeding your child using one breast, use the breast pump on the other breast at the same time.
- Warmth and relaxation encourage milk flow. Use the breast pump after having a shower or bath.
- If your breasts are swollen or sore, place a warm towel on them for a few minutes. This will improve the milk flow.
- If the milk does not start to flow from your breasts immediately, try to relax and then try again a few minutes later.
- If the breast pump still does not express any milk from your breasts after five minutes, stop using it and try again later.
- If you experience any sharp pains when using the breast pump, consult a doctor.

## 7.2 Operating the breast pump

<p>1. Before using the breast pump, wash your hands and breasts thoroughly.</p>	
<p>2. Sit with your upper body lent forwards slightly. You can support your back better by using a pillow.</p>	
<p>3. Then place the breast shield with silicone cushion gently on the breast. Make sure that the nipple is completely enclosed and there are no air pockets. To ensure proper function, the breast shield must be drawn firmly onto the breast. Use the manual pump as shown in the image.</p>	
<p>4. You can vary the pump intensity by selecting between the MIN/ MAX markings that can be seen. It is recommended that you start in the MIN position.</p>	

## 7.3 Storing the breast milk

Expressed milk can be stored in the fridge or freezer for a certain period of time.

- When storing breast milk, ensure that the breast pump and the bottles are disinfected before use. Only store breast milk in disinfected containers.
- If you wish to store breast milk, you should place it in the fridge or freezer immediately after expressing. Do not leave breast milk out at room temperature.
- If storing breast milk in the freezer, it is recommended that you mark the bottle with the date of expressing.

### **WARNING**

Be sure to observe the storage periods in the following table:

	<b>ROOM TEMPERATURE</b>	<b>REFRIGERATOR (approx. 3 - 5°C)</b>	<b>FREEZER (approx. -16°C)</b>
<b>Freshly expressed breast milk</b>	Can be kept for 6 hours	Can be kept for 24 hours (Do not store in the fridge door, as it is not sufficiently cool there)	Can be kept for 3 months
<b>Defrosted breast milk (that had been frozen)</b>	Use immediately	10 hours	Do not refreeze!

## 7.4 Feeding breast milk to the baby

### Note

You can use the supplied bottle with the silicone teat when feeding the breast milk to your baby. Alternatively, you can also express the breast milk with an NUK or Avent bottle and then feed it to your baby. (Corresponding NUK/Avent adapter included in delivery).

- Prior to feeding, ensure the bottle has been disinfected before filling it with the breast milk that has been stored.
- To defrost frozen breast milk, store it overnight in the fridge, so as to retain the nutrients. Once the breast milk is defrosted, it must be consumed within 24 hours. Breast milk should only be defrosted in warm water in especially urgent cases.
- If the breast milk smells unpleasant, it should be disposed of immediately.
- Never heat up breast milk in the microwave as it may be heated unevenly and could burn your child when they drink it.
- If heating up the breast milk, check its temperature before feeding it to your baby.

## 8. Accessories and replacement parts

Accessories and replacement parts are available from the corresponding service address (according to the service address list). Please state the corresponding order number.

<u>Item</u>	<u>Item number</u>
Bottle with teat and bottle cap	163.719
Adapter for Avent and NUK bottles	163.720
2 silicone cushions (small and standard)	163.721
Handle for manual pump including silicone cushion (standard size)	163.722
Bottle stand	163.723

## 9. Disposal

If the device is not to be used again, the parts can be disposed of with normal household waste.

## 10. What if there are problems?

<u>Problem</u>	<u>Potential solutions</u>
There is no suction from the breast pump.	<ul style="list-style-type: none"><li>• Check whether all parts of the breast pump are correctly assembled.</li><li>• Check whether the pump attachment is correctly placed on the breast.</li></ul>
No breast milk is being expressed.	<ul style="list-style-type: none"><li>• Ensure that there is no residual milk in the silicone valve obstructing the milk flow.</li></ul>
Expressing breast milk is causing pain.	<ul style="list-style-type: none"><li>• Try to relax and attempt it again. As you use the breast pump more and more, you will find it easier to express breast milk.</li><li>• Set the suction level to MIN.</li><li>• Consult your doctor.</li></ul>
The breast milk is leaking out underneath the silicone cushion.	<ul style="list-style-type: none"><li>• Remove the silicone cushion and re-attach it. Ensure the silicone cushion is correctly positioned and firmly secured to the pump attachment.</li><li>• Lean your upper body forwards slightly.</li></ul>

Problem	Potential solutions
One of the breast pump components has become altered or damaged.	<ul style="list-style-type: none"> <li>Do not use any corrosive, gaseous cleaning agents or solvents.</li> <li>In some cases, marked fluctuations in temperature can damage the plastic on the breast pump.</li> <li>If any of the parts becomes damaged, stop using the breast pump.</li> </ul>

## 11. Technical specifications

Model	BY 15
Dimensions	approx. 9.1 x 15.5 x 21 cm
Weight	135 g
Maximum suction pressure	-0.32 bar
Permissible operating conditions	+5°C to +40°C; 30% to 85% relative humidity (non-condensing); 700 – 1060 hPa ambient pressure
Permissible storage conditions	-20°C to +60°C; 10% to 95% relative humidity (non-condensing); 700 – 1060 hPa ambient pressure
Child use and care articles - Drinking equipment according to EN 14350	
Device for expressing breast milk	
Device with fine vacuum and low flow	

Technical information is subject to change without notification to allow for updates.

The device complies with the Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices, as well as the respective national provisions.

## 12. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.**

**The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.  
German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

For users/patients in the European Union and identical regulation systems (EU Medical Device Regulation (MDR) 2017/745), the following applies: If during or through use of the product a major incident occurs, notify the manufacturer and/or their representative of this as well as the respective national authority of the member state in which the user/patient is located.

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